


# RHINO UNIVERSAL SLINGS - STANDARD & FULLBACK



STANDARD

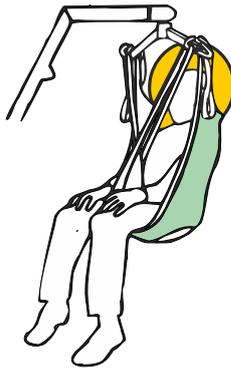

FULLBACK  
(With Head Support  
As Indicated in Yellow)

## The Rhino Universal Slings

The Slings are designed for use with most patients.

The Slings are available in 2 styles - **Standard and Fullback**.

The **Fullback** includes a Full Head and Shoulder Support.

## Product Features

### For Standard and Fullback

1 - Applicator Pockets - these facilitate the application of the Sling to the Patient.

2 - Modesty Strap (Leg Loop) - To prevent the Legs from "Yawning", feed the leg without strap (loop) into the leg of the Sling with a Loop.

### For Fullback Only


3 - The Head Support Spines are permanent - they are made of a special construction of materials and are sewn into the sling. No More Lost Spines.

4 - The Modesty Strap can also be used to convert the Fullback into a Amputee Sling - See Diagrams below.


5 - "Bucket Seat"

Standard Series

Fullback Series

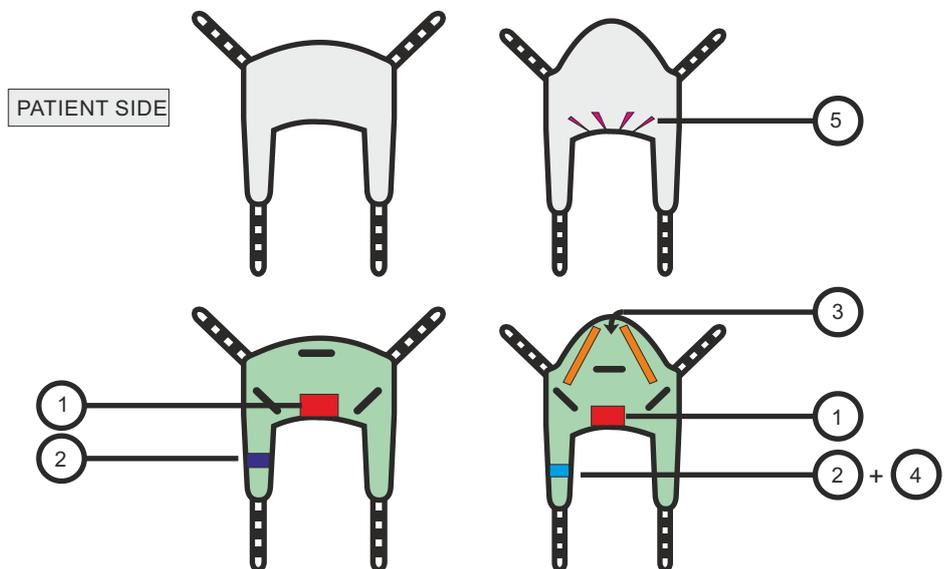


## Conversion of the Fullback Sling into a Amputee sling

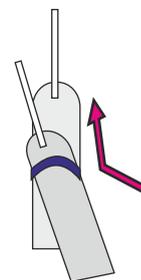


### MODESTY STRAP

To prevent the legs from "yawning", feed the leg of the sling without the Strap into the leg of the

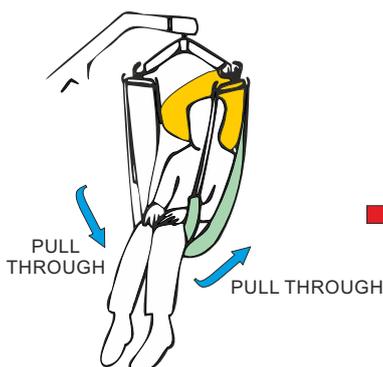

Apply the Leg of the Sling without the strap into the Leg of the Sling with the Loop and PULL Through.

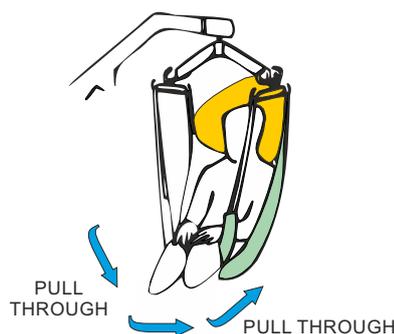

Adjust the Legs and attach the Sling Straps to the Hoist (Lifter).