

CARE SYMBOLS USED AND GENERAL LAUNDRY INFORMATION

General Laundry Information

Use Mild Soap Solution.

Wash at temperatures up to 80° C (Degrees Centigrade) or 176° F (Degrees Fahrenheit) and then rinse.

Do Not Use Biological Detergents.

Do Not Place On Convector Heaters or Steam Pipes.

The Washing Temperature of your Sling will depend on the type of Sling and the Padding used.

The “Fortrend” Range of Slings will allow washing temperatures of 100 Degrees Centigrade (212 Fahrenheit).

Please Contact Your Rhino Partner (Dealer) for further information.



Drip Dry



Dry Clean



Do Not
Tumble Dry



Read The Care
Instructions Booklet



Hang to
Dry



Do Not
Chlorine Bleach




Do Not
Iron



Do Not
Smoke



Tumble Dry
At Low
Temperature

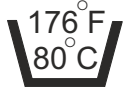

Washing
Temperature



Do Not
Chlorine Bleach



Washing
Temperature

Care Symbols

These are the same symbols on the Care Labels on the Patient Slings. The Colour and Size Information is as used on the Care Labels on the Sling.