

RHINO HAMMOCK STYLE (OR LONGSEAT) SLING

STANDARD



AS A AMPUTEE SLING



The Rhino Hammock Slings

The Slings are designed for use with most patients.

The Slings are available in 2 styles - with and without a Commode Opening.


The Hammock Slings are sometimes known as "Amputee" Slings because they are used extensively as such.

The Hammock Slings are also used with Patients with limited or no mobility and for Young Adults and Children.


We recommend the Rhino High Impact Material Construction for these types of Slings.



PLEASE NOTE: THIS SLING MAY ONLY BE FITTED FROM A LYING POSITION

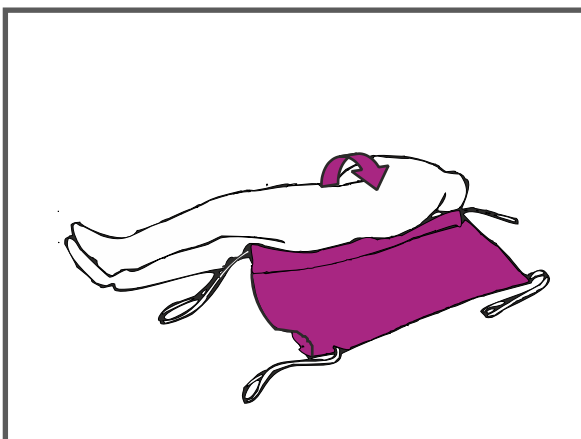

Draw Sheet Roll (or Log Roll) the patient onto the sling. With the Sling with a Commode Aperture, ensure the top of the Commode Aperture is at the base of the spine.

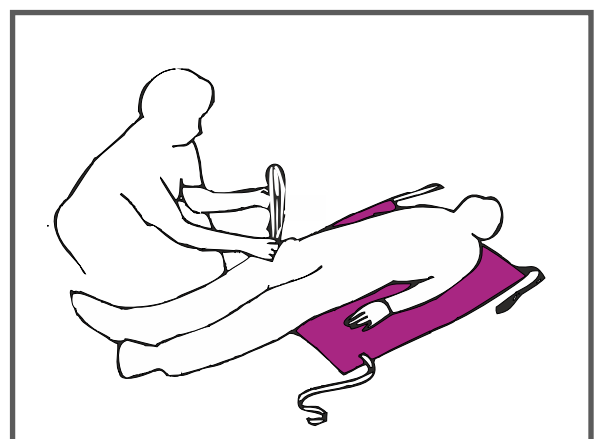

Ensure that the Sling is smooth under the patient. Gently pull the Webbing Straps through as indicated. Attach the Leg Displacement Straps to the Hoist (Lifter) first. Ensure the Patient's arms and elbows are placed within the confines of the sling - as shown in the diagram above.